

KLEAR^{Throat}



ARKONA

KLEAR THROAT

mouth and throat spray

 Medical device

Always use this product in accordance with the instructions for use. If in doubt, talk to your doctor or pharmacist.

KLEAR THROAT – a medical device designed to promote the treatment of throat inflammations caused by bacterial and viral infections. Intended for patients with impaired physiological function of the oral and pharyngeal mucosa. Has a protective effect on the mucous membrane of the mouth and throat. Restores the beneficial environment of the oral cavity that supports the body's natural defence mechanisms and facilitates regeneration of the pharyngeal mucosa.

MECHANISM OF ACTION

KLEAR THROAT moisturises and protects the mucosa of the mouth and throat.

This effect is ensured by:

- a combination of mineral salts with a composition similar to that of human saliva, to promote hydration of the mucosa,
- a phosphate buffer, to balance oral pH and maintain physiological pH levels of the saliva,
- xylitol – a natural sweetener, which stimulates the secretion of saliva to help maintain oral homeostasis and inhibit the growth of pathogenic microflora,
- a combination of glycerin and polysaccharides extracted from carrageen moss (carrageenans), which ensure optimal consistency of the product, increased viscosity, and a mucoadhesive and protective effect,
- preservation with a quaternary ammonium compound (domiphen bromide), which besides its preservative effect also works as a surfactant,
- peppermint flavour for a refreshing taste.

INDICATIONS FOR USE

KLEAR THROAT can be used to support treatment in:

- tonsillitis,
- pharyngeal mucositis,
- oral mucositis,
- recovery after tonsillectomy.

INGREDIENTS

purified water, mineral salts, carrageen moss (*Chondrus crispus*) extract, xylitol, glycerine, domiphen bromide, peppermint flavour.

EN



Fig. 1

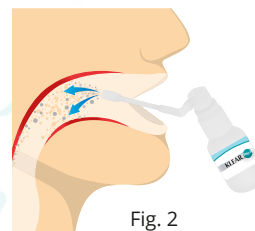


Fig. 2

CONTRAINDICATIONS

Hypersensitivity or allergy to any of the ingredients.

DIRECTIONS FOR USE

Shake before use.

Before first use: Holding the container upright, with the dosing tip facing away, prime the applicator by pressing it several times (until mist appears) (Fig. 1).

1. Place the applicator in your mouth and push it to spray 3-4 doses of the product into your throat (Fig. 2). After application, it is recommended to refrain from swallowing for as long as possible.
2. Repeat application as needed, 6-8 times a day.

KLEAR THROAT throat spray [can] be used long-term. The product does not cause dependency. It does not adversely affect driving or operating machinery and technical equipment.

ADVERSE REACTIONS

To date, no adverse reactions have been reported after using the product as recommended. However, as any other preparation, this product may cause adverse reactions. In the event of any discomfort, stop using the product and contact a medical professional without delay.

PRECAUTIONS

Do not unscrew the pump or disconnect the applicator. Avoid contact with eyes. Do not use after the expiry date.

STORAGE

Store at a temperature below 25°C. Do not overheat. Keep out of the reach of children.

CONTAINER

Bottle 20 ml (200 doses) with an atomiser and applicator.

Last revised: rev. 2, 28 Aug 2023

 **ARKONA**

ARKONA Laboratorium Farmakologii Stomatologicznej
Nasutow 99C, 21-025 Niemce, Poland, EU

