PERIO-FLUSH

re.v. 2, 18.05.2023

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EN
FR
DE
ES
IT
FR
DE
ES

PERIO-FLUSH

Solution for the irrigation of periodontal pockets in a syringe dispenser

[MD]Class I medical device

PERIO-FLUSH is a ready-to-use liquid solution for the irrigation and rinsing of periodontal pockets. It can be used to support non-surgical treatment of periodontal pockets. It effectively removes solid impurities from the pocket. It does not disturb the regeneration of connective tissue attachment.

MECHANISM OF ACTION

PERIO-FLUSH contains a solution of mineral salts with the addition of a quaternary ammonium compound (domiphen bromide). The main action of PERIO-FLUSH consists in mechanical removal of inflammatory content from the periodontal pocket.

INDICATIONS FOR USE

- Inflammation of periodontal pockets, in particular:
 - around class II fillings or areas with no contact points,
 - around teeth with difficult eruption, especially wisdom teeth,
 - around teeth intended for extraction,
 - around crowns,
 - around teeth supporting denture clasps, clips or removable braces,
 - around fixed orthodontic appliances,
- Soft tissue inflammation under restorations such as bridges;
- Soft tissue inflammation under dentures and removable braces;
- Rinsing the pockets after scaling.

GENERAL HANDLING INSTRUCTION

If possible, bring PERIO-FLUSH to body temperature to minimise patient discomfort during application. Unscrew the cap and mount the disposable applicator by screwing it on the luer-lock of the syringe. Make sure the dosing system is working correctly by gently pressing on the plunger above from the patient's mouth until a drop of the product can be seen. If it is impossible to extrude the product, replace the applicator with a new one. If the process is successfully completed, you can start working with the patient. To do that, gently insert the applicator down to the bottom of the periodontal pocket. Upon reaching the bottom, rock the applicator to the left and right from the bottom of the pocket and start irrigating by slowly pressing on the syringe plunger. Below you will find detailed instructions on how to proceed in different clinical cases. Continue until the liquid flowing out of the pocket is clear. After the procedure is finished, instruct the patient to spit out any remaining product and rinse their mouth with mouthwash, preferably MUSEON solution. Detach and dispose of the used applicator and the empty syringe according to the relevant regulations.

INSTRUCTIONS FOR USE

INFLAMMATION OF PERIODONTAL POCKETS:

- around class II fillings or areas with no contact points**
 - Apply PERIO-FLUSH to the inflamed area with mouthwash, preferably MUSEON solution. Remove solid plaque, taking care not to damage the root cementum. While irrigating, move the applicator around all the tooth roots, keeping the tip close to the bottom of the pocket. Go round each tooth and root separately, making sure that all the inflammatory content is washed out. Continue irrigating as long as there is any inflammatory content or tartar remains flowing out of the pocket.
- around teeth with difficult eruption, especially wisdom teeth**
 - Apply PERIO-FLUSH to the inflamed area with mouthwash, preferably MUSEON solution. Remove food debris with an explorer or another instrument. While irrigating, move the applicator around the tooth, keeping the tip as deep as possible. Go round the tooth, making sure that all the inflammatory content is washed out. Continue irrigating as long as there is any inflammatory content or tartar remains flowing out of the pocket.
- around teeth intended for extraction**
 - Apply PERIO-FLUSH to the inflamed area with mouthwash, preferably MUSEON solution. While irrigating, move the applicator around the tooth, keeping the tip close to the bottom of the pocket. Go round the tooth, making sure that all the inflammatory content is washed out. Continue irrigating as long as there is any inflammatory content or tartar remains flowing out of the pocket, and until the site where extraction instruments are to be inserted is clean.

around crowns

Apply PERIO-FLUSH to the inflamed area with mouthwash, preferably MUSEON solution. Remove food debris with an explorer or another instrument, also under the crown. While irrigating, move the applicator around the tooth and under the crown, keeping the tip as deep as possible. Go round the tooth, making sure that all the inflammatory content is washed out. Continue rinsing as long as there is any inflammatory content or tartar remains flowing out of the pocket.

around teeth supporting denture clasps, clips or removable braces

Ask the patient to rinse their mouth with mouthwash, preferably MUSEON solution. Remove food debris with an explorer or another instrument. While irrigating, move the applicator around the tooth, keeping the tip as deep as possible. Go round the tooth, making sure that all the inflammatory content is washed out. Continue irrigating as long as there is any inflammatory content or tartar remains flowing out of the pocket, and until the site where extraction instruments are to be inserted is clean.

around crowns

Apply PERIO-FLUSH to the inflamed area with mouthwash, preferably MUSEON solution. Remove food debris with an explorer or another instrument, also under the crown. While irrigating, move the applicator around the tooth and under the crown, keeping the tip as deep as possible. Go round the tooth, making sure that all the inflammatory content is washed out. Continue rinsing as long as there is any inflammatory content or tartar remains flowing out of the pocket.

around fixed orthodontic appliances

Apply PERIO-FLUSH to the inflamed area with mouthwash, preferably MUSEON solution. Remove solid plaque, taking care not to damage the root cementum. Move the bracket to a safe distance from the gingiva (an additional 2 mm is usually enough). While irrigating, move the applicator around the tooth, keeping the tip close to the bottom of the pocket. Go round the tooth, making sure that all the inflammatory content is washed out. Continue irrigating as long as there is any inflammatory content or tartar remains flowing out of the pocket.

around teeth with difficult eruption, especially wisdom teeth

Apply PERIO-FLUSH to the inflamed area with mouthwash, preferably MUSEON solution. Remove solid plaque, taking care not to damage the root cementum. Move the bracket to a safe distance from the gingiva (an additional 2 mm is usually enough). While irrigating, move the applicator around the tooth, keeping the tip close to the bottom of the pocket. Go round the tooth, making sure that all the inflammatory content is washed out. Continue irrigating as long as there is any inflammatory content or tartar remains flowing out of the pocket.

SOFT TISSUE INFLAMMATION UNDER RESTORATIONS SUCH AS BRIDGES

Ask the patient to rinse their mouth with mouthwash, preferably MUSEON solution. Use an explorer or another instrument to remove food residue from the spaces between the pontic and the gingiva, taking care not to press too hard on the gum. Insert the applicator under the pontic from the side of the oral cavity proper and start flushing the applicator along the contact from one abutment tooth to the other, taking care to keep the applicator tip as deep as possible. If necessary, insert the applicator from the vestibular side and continue flushing until all the inflammatory content is washed away. Continue irrigating as long as there is any inflammatory content or solid impurities flowing from under the bridge.

SOFT TISSUE INFLAMMATION UNDER DENTURES AND REMOVABLE BRACES

Ask the patient to rinse their mouth with mouthwash, preferably MUSEON solution. Adjust temporarily the base of the denture or clasps, removing points of pressure on the mucosa. Make sure the surface of the base of the denture or braces is smooth. Gently massage the surface, filling in a new prosthetic restoration. Flush the swollen gingival surface.

RINSING THE POCKETS AFTER SCALING

Ask the patient to rinse their mouth with mouthwash, preferably MUSEON solution. Remove solid plaque, taking care not to damage the root cementum. While irrigating, move the applicator around the tooth, keeping the tip as deep as possible. Go round each contact, rinse with plenty of water. In the event of an allergic reaction, stop using the product. If necessary, consult an expert.

PRECAUTIONS FOR DENTAL PERSONNEL

Always wear personal protective equipment such as gloves, a face mask and safety glasses. Avoid contact with eyes. In case of contact, rinse with plenty of water. In the event of an allergic reaction, stop using the product. If necessary, consult an expert.

PRECAUTIONS FOR DENTAL PERSONNEL

Always wear personal protective equipment such as gloves, a face mask and safety glasses. Avoid contact with eyes. In case of contact, rinse with plenty of water. In the event of an allergic reaction, stop using the product. If necessary, consult an expert.

PRECAUTIONS FOR DENTAL PERSONNEL

Always wear personal protective equipment such as gloves, a face mask and safety glasses. Avoid contact with eyes. In case of contact, rinse with plenty of water. In the event of an allergic reaction, stop using the product. If necessary, consult an expert.

PRECAUTIONS FOR DENTAL PERSONNEL

Always wear personal protective equipment such as gloves, a face mask and safety glasses. Avoid contact with eyes. In case of contact, rinse with plenty of water. In the event of an allergic reaction, stop using the product. If necessary, consult an expert.

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