



Created by Dentists for Dentists



ARKONA

Tooth sealing 17 SEALPRIM and ETCHGEL patient 18 years old



QUICK AND EFFECTIVE SOLUTIONS TO SERIOUS PROBLEMS




www.arkonadent.com



CASE DESCRIPTION



ARKONA

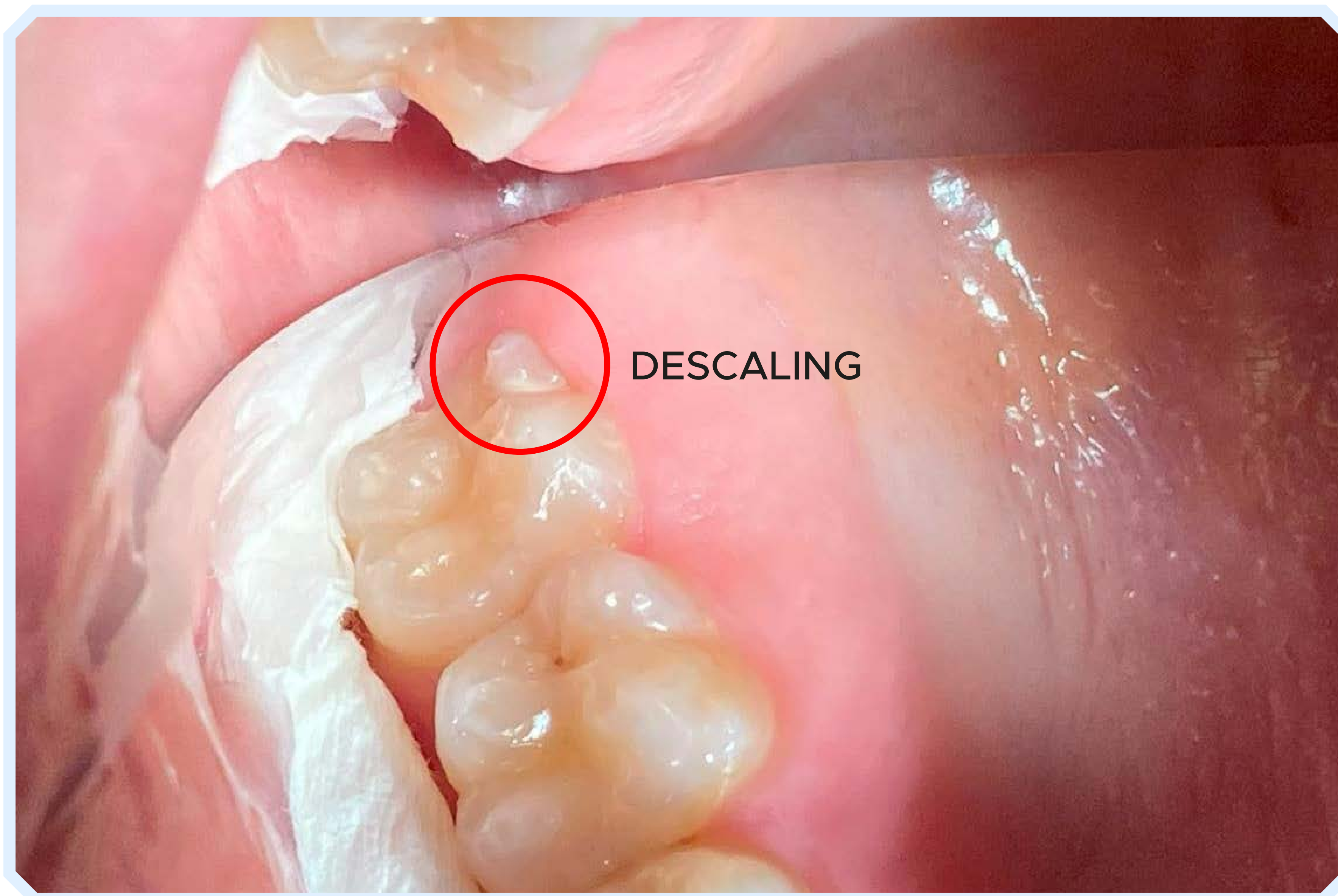

- A frequently observed phenomenon, especially in our younger patients, is the appearance of decalcifications, indicating the initial stage of caries.
- We usually notice them in hard-to-reach places
- of the oral cavity, i.e. grooves and pits of the lateral teeth. We see such a situation in our 18-year-old patient.





STAGE 1



ARKONA



- We apply **ETCHGEL** to the cleaned and dried surface of the pits and fissures, which prepares the enamel surface for the application of **SEALPRIM**.
- Thanks to the gel consistency of the material, we are able to precisely distribute the product, without causing uncontrolled demineralization of the enamel areas we do not want.
- The etched enamel surface is chalky white, which makes it easier for us to precisely apply the sealant to the tooth surface. Its liquid consistency ensures comfortable work, and the disposable applicator ensures the safety of our patients and increases the standard of services provided.





STAGE 2



ARKONA



- After polymerizing the material, we have a tooth protected against caries. The entire process takes only 15 minutes! Sealing saves our time and makes work easier. We save patients stress and pain, thanks to which we gain their trust and gratitude from parents for protecting their children's teeth.





WHY IS IT WORTH USING SEALING?



ARKONA



- You can easily and quickly perform the procedure
- Non-invasive, low-cost procedure
- Less stress for you related to your patient's „suffering“
- A satisfied, enthusiastic client is your profit!



www.arkonadent.com



ARKONA

Created by Dentists for Dentists



www.arkonadent.com