



**ARKONA**

Created by Dentists for Dentists

# Construction of a bridge based on the **COMCORD** structure on teeth 42-31

 QUICK AND EFFECTIVE SOLUTIONS TO SERIOUS PROBLEMS

ETCHGEL • NEXT • GLAZE



[www.arkonadent.com](http://www.arkonadent.com)



## CASE DESCRIPTION



**ARKONA**



- Missing teeth are a common health problem that concerns not only dental issues, but also psychological ones, related to shame and a sense of exclusion due to incomplete dentition, especially when this fact cannot be hidden.
- In the case of missing teeth concerning incisors, we often deal with mechanical damage, as a result of accidents. In such cases, it is crucial for the patient to fill this gap as soon as possible.
- An excellent alternative to prosthetic filling of a missing tooth is the use of **COMCORD** to create an adhesive bridge. This simple procedure, which you will perform during one visit, will restore your patient's dream smile!







ARKONA



## STAGE 1 - PREPARATION FOR THE PROCEDURE AND CONSTRUCTION OF THE ADHESIVE ARCH



- After etching the prepared areas in the adjacent teeth, the final restoration is made based on the previously measured **COMCORD** fragment. After securing the gum with a celluloid strip and polymerizing the material, we restore the missing tooth using the NEXT composite.



[www.arkonadent.com](http://www.arkonadent.com)





## STAGE 2 - GLAZE



**ARKONA**



- After adjusting the restoration in occlusion, the last step is to cover the restoration with **GLAZE**. You will get the effect of a polished and shiny tooth in a few seconds!



[www.arkonadent.com](http://www.arkonadent.com)



YOUR BENEFITS OF USING **COMCORD**



**ARKONA**



- **Single-visit filling of missing teeth**
- **Satisfaction of patient needs** - gratitude and satisfaction
- **Easy correction** - good aesthetics and functionality of the tooth for many years
- **Saving time of the procedure** - your profit
- **Non-invasive possibility of filling missing teeth**
- **Possibility of curing patients**, whom we have lost for various reasons so far



[www.arkonadent.com](http://www.arkonadent.com)





# ARKONA

**Created by Dentists for Dentists**



[www.arkonadent.com](http://www.arkonadent.com)