



ARKONA

Created by Dentists for Dentists

COMCORD based bridge on teeth 13-22

 QUICK AND EFFECTIVE SOLUTIONS TO SERIOUS PROBLEMS

ETCHGEL • MASTERBOND • GLAZE • CREATE



www.arkonadent.com



CASE DESCRIPTION



ARKONA



- The photo shows the initial state.

- Missing teeth in the aesthetic part of the jaw are a significant problem among dental patients. Neglect of hygiene and injuries cause tooth loss, which results in serious complexes and a lack of desire to smile in people affected by this problem.
- Obstacles to regaining the old smile are often fears, finances and time. In such situations, it is worth finding a solution that will be adapted to the possibilities and needs of the Patient.
- In the case of this Patient, the use of **COMCORD** was proposed to create a combined adhesive restoration.
- The patient will be able to enjoy a dazzling smile after just one visit!





STAGE 1 - PREPARATION



ARKONA



- Before starting work, it is necessary to prepare carious tissues and then etch these surfaces thoroughly. **ETCHGEL** etches enamel for 30 seconds and dentine for 10 seconds. Rinse thoroughly, dry and cover with the **MASTERBOND** bonding system to create **COMCORD** adhesion with tooth tissues. Such preparation of the work area will provide us with comfort of work and significantly extend the durability of the adhesive restoration.





STAGE 2 - CONSTRUCTION OF THE ADHESIVE ARCH AND RECONSTRUCTION OF MISSING TEETH



ARKONA



- After appropriate preparation of the areas prepared in the arch using **COMCORD**, we make a ring-shaped front teeth base.
- After its polymerization, we rebuild the missing parts of the teeth using **CREATE** composite.
- Teeth reinforced in this way are resistant to forces much stronger than chewing forces.





STAGE 3 - AESTHETIC FINISHING



ARKONA



- After calcifying and adjusting the teeth in the bite, it is time to give the work a shine. We do this step using **GLAZE**.
- Its use saves a lot of time on mechanical polishing of the work and thanks to the shine it visually rejuvenates the smile.
- The final effect of the aesthetic and durable work reinforced with **COMCORD**.





BENEFITS OF USING COMCORD



ARKONA



- **Single-visit filling of missing teeth**
- **Satisfaction of patient needs** - gratitude and satisfaction
- **Easy to make corrections** - good aesthetics and functionality of the tooth for many years
- **Saving time of the procedure** - your profit
- **Non-invasive possibility of filling missing teeth.**
- **Possibility of curing patients**, whom we have lost for various reasons so far





ARKONA

Created by Dentists for Dentists



www.arkonadent.com