



## PERIO-FLUSH

**For everyday battles  
against periodontal  
disease**

**PERIO-FLUSH** is a professional solution **for rinsing periodontal pockets**, designed for everyday use in dental practice. It effectively cleanses periodontal pockets, reduces the load of anaerobic bacteria, and helps limit bleeding and inflammation, **creating better conditions for periodontal tissue regeneration**. By combining a gentle, isotonic formula with the precise application of a syringe, it provides **full control** over the procedure and enables **effective irrigation** even in the most demanding, hard-to-reach areas.

## Why PERIO-FLUSH?

**Non-foaming**



**It does not foam** during application, improving patient comfort and allowing the clinician to maintain full visibility and control throughout the procedure.

**Chlorhexidine-free**



**The formula is based on domiphen bromide** and contains no added chlorhexidine, making it gentle on tissues and well tolerated even in inflammatory conditions.

**Antibiotic-free**



**It contains no antibiotics**, eliminating the risks associated with overuse and the development of bacterial resistance.

**For professional use**



**Ideal for daily in-office procedures** – it works quickly, usually within 2–3 minutes, and can be seamlessly combined with many standard hygiene and periodontal treatments.

## Clinically proven effectiveness\*

98%

**PERIO-FLUSH** effectively removed deposits from 98% of irrigated periodontal pockets.

1,3 mm

On average, the examined group of periodontal pockets showed a reduction in pocket depth of 1.3 mm after treatment with **PERIO-FLUSH**.

79%

In 80.4% of the examined periodontal pockets, irrigation with **PERIO-FLUSH** reduced the anaerobic bacterial load, by an average of 79%.



Irrigation with **PERIO-FLUSH** had a statistically significant effect on reducing bleeding.

\*Clinical study: UR.DNB.4501.0001.2018



BEFORE




AFTER

## Indications of use


- **After scaling and tartar removal** – irrigation to flush out remaining deposits and inflammatory content
- **Difficult eruption (e.g. wisdom teeth)** – cleansing painful, hard-to-reach periodontal pockets
- **Interdental spaces** – especially where proper contact points are missing
- **Around orthodontic appliances** – improved hygiene around brackets and archwires
- **Before extraction** – cleansing the operative field and reducing the risk of complications
- **Around crowns, bridges and denture clasps** – regular control of inflammation in hard-to-access areas

## Examples of clinical use

- ◆ Irrigation of periodontal pockets after tartar removal

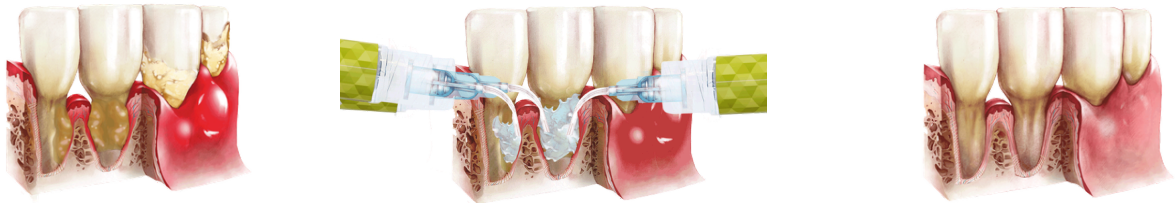

- ◆ Around Class II restorations when proper contact points are missing

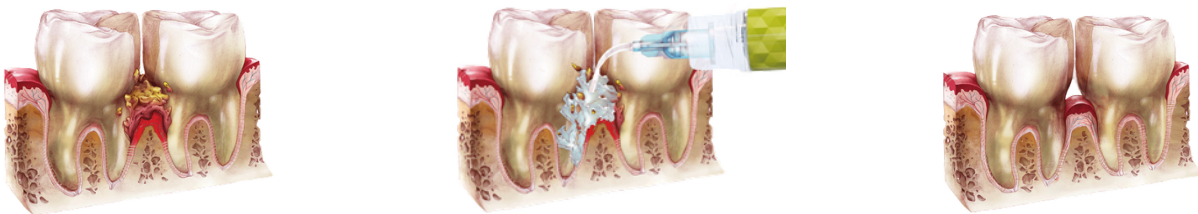

- ◆ Around fixed orthodontic appliances



## Important information

- ? **Why not a gel?**  
Because gel remains in the pocket and may hinder reattachment. A solution irrigates and rinses the pocket without leaving residue behind.
- ? **How often should the procedure be repeated?**  
Every 1–3 months, until the pocket depth is reduced to below 5 mm.
- ? **Is PERIO-FLUSH suitable for home use?**  
No. It is a professional in-office procedure – which is why it is effective.
- ? **Can it be used in children and pregnant women?**  
Yes, provided the person treated has no allergy to any of the ingredients.
- ? **How quickly can improvement be seen?**  
In most cases – after the first irrigation.
- ? **What does PERIO-FLUSH taste like?**  
The taste is distinctly salty, with a subtle apple-mint note.

## Package

- **Set of 10 syringes (3 ml each) + 10 applicators**

